

Technical Parameter

Voltage: 220V 50-60HZ
Motor: 12VDC 8.4A
Frequency: 50W

Shoe Cover Dispenser

Operation Manual



Catalogue

1. Function & Advantage

2. Control panel

3. Using Step


4. Notice

5. Maintenance & Repair

Maintenance & Repair
All machine have been tested okay before delivery, if occur below problem, please check as follows:

Error	Reason	Solution
Machine don't work	Have electric circuit Errors.	Please check electric circuit
Hook can't catch shoe covers from Box	Shoe covers use up	Change new shoe cover box
	Shoe covers are rambling	Lightly arrange the covers through the hole of the box, make sure the center of shoe cover's elastic is in the middle of the box's exit.
Shoe cover can't outspread or only can be hooked one angle	The haul hook dislocation	Please release hook's screw, adjust the haul hook, then fix screw.
The catch hook or the haul hook stop and warning	Overtravel-limit switch error	Please check all the overtravel limit switch, make sure switch well.
The shoe cover is dispensed successfully, suddenly the machine begin warning.	the upper right overtravel limit switch error.	Check the switch whether is closed after dispensing shoe cover.
The shoe cover was used, but the machine can't continue to dispenser next one	the upper right overtravel limit switch error.	Check the switch whether is open.
Electric circuit is regular, but machine don't running, the face light twinkle.	Photoelectric switch error. Infrared lamp or infrared receiver error	Please check photoelectric switch and the electric circuit. Please check the infrared is exactly launched into receiver's hole.

1. Shoe cover storage box _____
2. Warning light _____
3. Sterilizing Light _____
4. Counting Monitor _____
5. Power switch _____
6. Count righting switch _____
7. Ultraviolet radiation switch _____
8. Stance _____
9. Housing _____

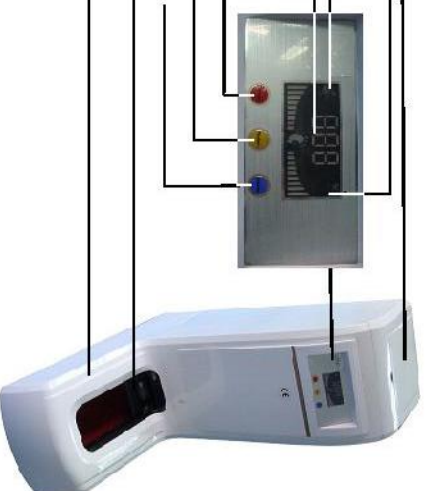

Attation:

1. Highlight can infrared lamp, please avoid use machine in highlight surroundings.
2. When adjust the infrared lamp, please have sunglasses for protecting eyes.
3. Please lubricate the machine in time.

**PLEASE READ THIS OPERATION
MANUNAL BEFORE USING MACHINE**

Using Steps:

- 1、Open the cover of shoe cover box and put the plastic weigher into box (see drawing No.02)



No.02

- 2、Open the upside cover of the machine, insert the plastic shoe cover box into the box base and then close the cover. (see drawing No.03)



No.03

- 3、After power up, then press the button of “off/on”, machine will start to take out shoe cover. (see drawing No.05)



No.04

- 4、Step into the hole of machine uprightly to get shoe cover (see drawing No.05)



No.05

- 5、After 1pcs shoe cover was stepped out, machine will start to take out and open the next 1pcs shoe cover and will repeat the same action until the box shoe cover be finished. the digital screen will come down one by one till to “0”. repeat the action of step one. After change each box of the shoe cover, please press the button of “Diagnosis”, it will reset the digital to be “200”.


Notice:

- 1、 According to different area, please make sure the voltage is 220V and the frequency is 50 HZ.
- 2、 The number showed in digital screen is the current remaining shoe cover quantity, don't press the button of "Diaplasis". Otherwise, the digital will be reset to "200".
- 3、 Don't put shoe into machine hole for stepping shoe cover, until the shoe cover be complete take out and opened. or machine will stop working and the error light is lighting and giving warning alarm.
- 4、 If something drops into machine hole when the machine is working, machine will stop working immediately and the yellow light will light with alarm. Now, you just need to press "OFF" firstly and remove this dropped thing is ok, and then the machine will work properly again by press "ON".
- 5、 When step the shoe cover, please keep your shoe is upright to machine hole, if one of the plastic shoe cover corners is hooked by machine because of your mistake, please press "OFF" and take the shoe cover out and then press on again.
- 6、 If machine can give shoe cover properly or no shoe cover come out, please take the shoe cover box out from machine and check the "O" rubber mouth is in the center of the shoe cover box, if not, please adjust them in the center of box and then machine can working well again.
- 7、 In order to make all the shoe cover could be smoothly come down, please put the special plastic weighter into shoe cover box.
- 8、 Children can use this machine only under adult's conduct.

Function and Advantages:

1. High efficiency and convenience to change shoe cover boxes within 5 seconds:
2. Max. 200pcs plastic shoe cover capacity:
3. Intelligitized automatic technologies:
4. Giving warning alarm when machine breakdown.

Control panel

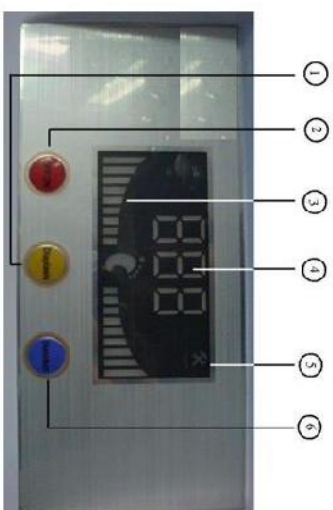

No.01

- | | |
|---------------------------|-------------------------------|
| 1、 Digital setting button | 2、 Switch "on / off" button |
| 3、 using Indicator lights | 4、 Balance shoe cover showing |
| 5、 Error/repair showing | 6、 Ultraviolet light |